

Accu-Temp Slide[®], U.S. Patent Number 10,431,126

Instructions For Use

Item Number: ACCU-10

APPLICATION

Accu-Temp Slides are designed to validate the slide-pad heating functionality of your advanced staining machine to ensure critical patient tissue slides stain optimally.

SUMMARY

Our U.S. patented Accu-Temp Slides are used to test the temperature threshold of each slide-pad on your machine. The slide features two temperature threshold indicating areas, one indicating 93°C and the other indicating 99°C, which will change from light gray to black when the indicated temperature has been reached.

INSTRUCTIONS

1. Load Accu-Temp Slides onto all 20 or 30 slide-pad numbered positions within your automated IHC or Special Stains machine.
2. Select your manufacturer recommended software maintenance procedure "Test-Temp Verify" described in your *Operator Guide*.
3. Allow slides to cool before removing from the pads -or- removing temperature labels from each slide.

ADDITIONAL FEATURES

Our water-resistant Accu-Temp labels feature a field to annotate each corresponding slide-pad number for passing or failing results. Each label may be removed from the slide starting with the top-right red corner and re-adhered to the 'Log Assist' package insert for accurate future reference or to trend problematic slide-pads.

Potential Results

